ACHILLODYN®

SPORLASTIC®

REF 07071
ACHILLODYN®

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INSTRUCTIONS FOR USE EN

Dear patient,

we ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

INTENDED USE

ACHILLODYN® is a compression support to relieve Achilles tendon pain by massaging and relieving friction pads.

INDICATIONS

- Conservative / postoperative
- Achillodynia (tendinopathy) with / without partial rupture
- Paratendinitis, paratendinopathy
- Bursitis subachillea
- Bursitis calcanea

KONTRA-INDICATIONS

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be supplied
- Impaired circulation or lymphatic soft tissue swelling
- Neurogenic disorders of the sensory system and skin trophism in the body area to be treated (sensory disorders with or without skin damage)
- Severe arterial circulatory disorders

SIDE EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

PUTTING IT ON AND USE

- ACHILLODYN® can be worn on the left or right side.
- Fold the upper edge of the support over to the end of the SPORLASTIC flag or to the upper orange stripe 1.
- Pull the support over the foot until the heel sits correctly in the anatomically knitted heel section 2.
- When properly seated, the pads lie in the pits to the side of the Achilles tendon. The pockets for these pads are deliberately kept slightly larger so that the pads can be moved slightly in height for an ideal fit 3.
- Fold the upper edge back again and pull the support into the correct position 4.
- ACHILLODYN® fits correctly when the seam runs centrally over the sole of the foot and heel 5.
- ACHILLODYN® should not be too tight.
- The two heel wedges should be at the very back of the shoes 6.

ACCESSORIES / SPARE PARTS

Optionally available from your specialist store:

- A pair of heel wedges (REF 17071) for ACHILLODYN®

IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- If a significant deterioration in health occurs during the use of the medical device, we ask you to report this serious incident to your specialist dealer, your doctor or us as the manufacturer, as well as to the responsible authority. You can find our contact information in these instructions for use.
- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- With products involving direct skin contact with an elastomer (e.g. a silicone pad), blistering may develop on the skin if the product is worn too tightly or if it is moved over the skin with high amounts of friction. In this case, we recommend stopping use of the product. Sweat can develop underneath the silicone, which in rare cases can trigger a reaction to your own sweat with corresponding allergic skin changes. SPORLASTIC is not aware of any allergic reactions being caused by the material. If you have any problems, please talk to your doctor or the supplier of the aid.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

CARE

We recommend washing ACHILLODYN® gently in lukewarm water (30°C) with mild detergent and air drying. Do not dry the product in a tumble dryer, on the heating or in the microwave.

Any tension in the knitted fabric that decreases over time will be restored by washing. We therefore recommend washing the support more frequently.

TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product

at a normal temperature and humidity.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

MATERIAL COMPOSITION

67% polyamide, 33% elastane

An online version of the instructions for use can be found on our website at www.sporlastic.de.

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