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## INSTRUCTIONS FOR USE EN

Dear patient, We ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

## INTENDED USE

COXA-HiT® is a hip support used for the conservative or postoperative treatment of hip joint disorders.

## INDICATIONS

- Post-operative, e.g. after hip operations (endoprotheses, installation or replacement)
- Conservative, e.g. osteoarthritis, muscle strains

## CONTRA-INDICATIONS

- Prolonged, rigid immobilisation, especially in the elderly.
- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be cared for.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenic disorders of the sensory system and skin trophism in the body area to be treated (sensory disorders with or without skin damage).

## SIDE EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

## INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

## PUTTING IT ON AND USE

- Open the Velcro fasteners of the support.
- Loosely attach the upper waist belt to the side of the large fleece on the leg section.

- The waist belt of the support is placed about 5 cm below the navel.
- While doing so, reach with your fingers into the gripping aid provided at the end of the hip straps.
- The support is applied in such a way that a comfortable compression is created. Now take the upper hip belt that has been loosely fastened at the side and pull it over the belt underneath with a strong pull. The silver-coloured compression control strip should be under full tension **1** (COXA-HiT® applied too loosely **2**).

- To put the leg section in place, use the integrated gripping aid on the inner thigh strap.
- The thigh belt is now closed circularly.
- The SPORLASTIC compression control strip attached here is fully tensioned when correctly applied.

- The enclosed cold/hot pad is used as directed by the doctor for thermotherapy and to increase the compression effect.

- Please follow the application instructions on the enclosed cold/hot pad.

- SPORLASTIC compression control strip:

- 1: COXA - HiT® applied with tension.
- 2: COXA - HiT® applied too loosely.

With the compression control strip, the doctor, patient or nursing staff can easily see whether the hip support has been put on functionally.

- In postoperative use, COXA-HiT® can be combined with an anti-thrombosis stocking.

## ⚠ IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sporlastic.de to report the incident.
- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.
- Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.
- SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

## CARE

We recommend washing COXA-HiT® gently in lukewarm water (30°C) with mild detergent and air drying. Do not dry the product in the tumble dryer, on the heating or in the microwave. Close the Velcro fasteners before washing. These will remain functional longer and damage to other garments can be avoided.

## ☂ ☀ TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

## QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

☑ An online version of the instructions for use can be found on our website at [www.sporlastic.de](http://www.sporlastic.de).

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