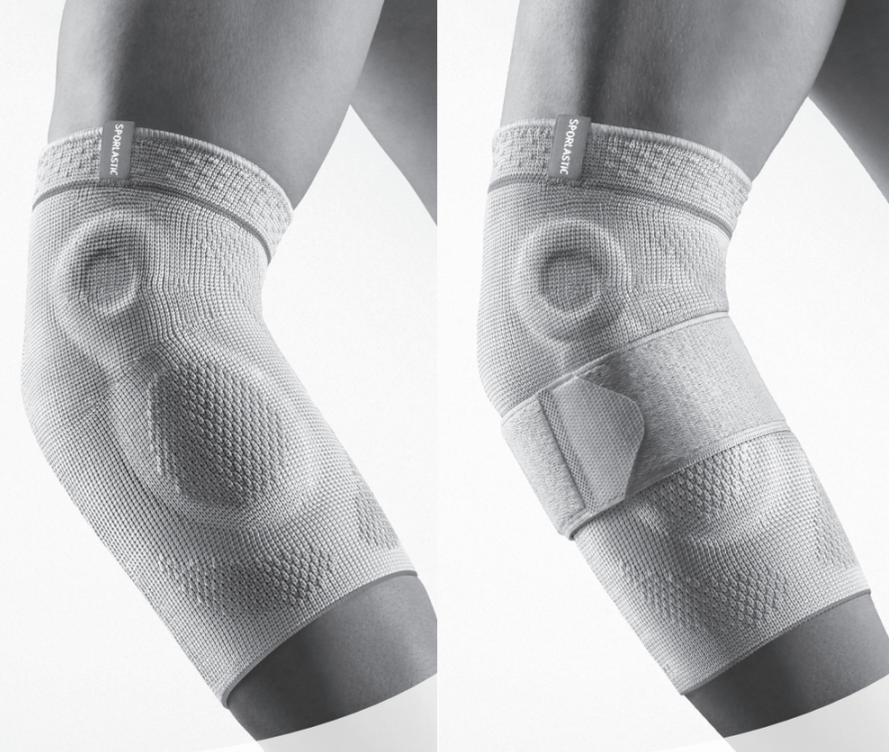
# EPIDYN®

# EPIDYN® AKTIV

**REF** 07237  
EPIDYN®

**REF** 87237  
EPIDYN® KIDS

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## INSTRUCTIONS FOR USE EN

Dear patient,

we ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

### INTENDED USE

EPIDYN® / EPIDYN® KIDS / EPIDYN® AKTIV is an elbow brace for relieving and stabilising the elbow joint.

### INDICATIONS

**REF** 07237, EPIDYN® / **REF** 87237, EPIDYN® KIDS:

- Conservative / postoperative
- Arthrosis
- Ligament laxity
- Elbow sprain
- Inflammatory conditions (e.g. after fractures and operations)
- Rheumatological conditions

**REF** 07515, EPIDYN® AKTIV:

- Conservative / postoperative
- Epicondylitis humeri radialis/ulnaris (tennis elbow, golfer's elbow)
- Arthrosis
- Ligament laxity
- Elbow sprain
- Inflammatory conditions (e.g. after fractures and operations)
- Rheumatological conditions

### CONTRA-INDICATIONS

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be supplied
- Impaired circulation or lymphatic soft tissue swelling
- Neurogenic disorders of the sensory system and skin trophism in the body area to be treated (sensory disorders with or without skin damage)

### SIDE EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

### INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

### PUTTING IT ON AND USE

- EPIDYN®, EPIDYN® KIDS and EPIDYN® AKTIV can be worn on the left and right.
- For EPIDYN® AKTIV, open the circular band.
- Pull the support over your elbow. The orange SPORLASTIC flag must be on the upper arm, the seam in the middle on the back of the elbow.
- Position the ring-shaped recesses of the silicone profile inserts so that they come to rest on the two lateral joint projections (epicondylus lateralis and medialis).
- With EPIDYN® AKTIV, fix the band circularly below the elbow. First guide the band from the inside over the upper side of the arm.
- Dose and vary the pressure intensity of the band individually according to your needs.

### ⚠ IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- If a significant deterioration in health occurs during the use of the medical device, we ask you to report this serious incident to your specialist dealer, your doctor or us as the manufacturer, as well as to the responsible authority. You can find our contact information in these instructions for use.
- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- Product may contain metallic objects. We recommend not putting them in contact with electromagnetic interference.
- With products involving direct skin contact with an elastomer (e.g. a silicone pad), blistering may develop on the skin if the product is worn too tightly or if it is moved over the skin with high amounts of friction. In this case, we recommend stopping use of the product. Sweat can develop underneath the silicone, which in rare cases can trigger a reaction to your own sweat with corresponding allergic skin changes. SPORLASTIC is not aware of any allergic reactions being caused by the material. If you have any problems, please talk to your doctor or the supplier of the aid.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

### CARE

We recommend washing EPIDYN®, EPIDYN® KIDS and EPIDYN® AKTIV gently in lukewarm water (30°C) with mild detergent and air drying. Do not dry the product in the tumble dryer, on the heating or in the microwave.

Tension in the knitted fabric that decreases over time is restored by washing. We therefore recommend washing the support more often.

### ☂☀ TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

### QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

### MATERIAL COMPOSITION

67 % polyamide, 33 % lycra

An online version of the instructions for use can be found on our website at [www.sporlastic.de](http://www.sporlastic.de).

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