FOOT-UP®

FOOT-UP® BARFUSS

 SPORLASTIC®

REF ART.-NR. 07810
FOOT-UP®

Textile Fußheberorthese

Textile foot flexor brace

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SIDE EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

PUTTING IT ON AND USE

FOOT-UP®, Art.No. 07810:

- We recommend wearing a sock under the product.

- FOOT-UP® can only be inserted and used in lace-up shoes.
- Detach the insole from the ankle strap by pressing the button in the middle of the quick release fastener 1.
- Put on the ankle strap with the narrow Velcro strip at the bottom and the quick-release fastener at the front. Place the ankle strap over the widest part of the ankle. Tighten the ankle strap but do not constrict the ankle 2.
- Place the insert between the flap and the lace 3. Then pull the elastic band between the two pieces of lace 4. Pulling the insert further towards the top of the foot will provide more support when lifting the foot. Make sure that the elastic band is also pulled further down the foot between the lace parts.
- Connect the ankle strap and the insole by pushing the two parts of the quick release fastener together 5.

FOOT-UP® BARFUSS, Art.No. 27810:

- The barefoot support is applied directly behind the ball of the foot 6. The support part with quick release fastener is positioned on the back of the foot and fastened.

- If the ankle band of the basic FOOT-UP® is already in place, the „barefoot“ support can simply be connected to the quick-release fastener of the ankle band via the plug-in fastener 7.

- To put on the ankle band, please compare the information above under putting on FOOT-UP®.

IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.

- Check the functioning of the product with your doctor or specialist retailer.

- The product is designed to treat one patient.

- Do not wear the product directly against open wounds.

- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.

- The product can come into contact with splash water.

- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.

- In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sporlastic.de to report the incident.

- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.

- Product may contain metallic objects. We recommend not putting them in contact with electromagnetic interference.

- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

- Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

- SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

CARE

We recommend washing FOOT-UP® products gently in lukewarm water (30°C) with mild detergent and air drying. Do not dry the product in the tumble dryer, on the heater or in the microwave. Remove the transparent shoe insert before washing the ankle strap. This can be cleaned with a damp cloth. Make sure to close the Velcro fasteners before washing!

TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

An online version of the instructions for use can be found on our website at www.sporlastic.de.

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