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INSTRUCTIONS FOR USE EN

Dear Patient,

We ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

INTENDED USE

GENU-HIT® GS is a brace to support the physiological guidance of the knee joint.

INDICATIONS

- Mild ligament insufficiencies, strains, ruptures
- Gonarthrosis
- Postoperative and post-traumatic irritations

CONTRA-INDICATIONS

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be supplied.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenic disorders of the sensory system and skin trophism in the body area to be treated (sensory disorders with or without skin damage).

SIDE EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

PUTTING IT ON AND USE

- Pull GENU-HIT® GS over the knee so that the kneecap is surrounded by the silicone profile insert without pressure. The two knee splints should not exert any uncomfortable pressure on the leg.
- Then pass the four reins of the brace through the respective loop and loosely fasten them.
- The rein above the calf is tied first with a firm but still comfortable pull. This is followed by the front reins below and above the kneecap. The upper back rein is sewn on at a slight angle. The course of the reins is adapted to the conical shape of the thigh.
- When applying and tucking this rein, make sure that the course is optimal without forming pockets or pressure on the thigh.
- The circular pressure of the four reins, together with the two joint bars, supports the passive stabilisation as well as guidance of the knee and at the same time ensures an optimal fit of GENU-HIT® GS.

NOTES FOR THE ORTHOPAEDIC TECHNICIAN

- The professional and individual fitting of the two joint splints **1** is decisive for good knee guidance and stabilisation of the knee as well as an optimal fit.
- Before putting on GENU-HIT® GS, remove the two joint bars through the buttonhole sewn into the lateral guide pockets.
- The two joint splints must be shaped to the individual anatomy of the patient's leg.
- Then insert them again through the buttonhole into the side pockets so that the short joint leg points downwards.

▲ IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sporlastic.de to report the incident.
- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- Product may contain metallic objects. We recommend not putting them in contact with electromagnetic interference.
- With products involving direct skin contact with an elastomer (e.g. a silicone pad), blistering may develop on the skin if the product is worn too tightly or if it is moved over the skin with high amounts of friction. In this case, we recommend stopping use of the product. Sweat can develop underneath the silicone, which in rare cases can trigger a reaction to your own sweat with corresponding allergic skin changes. SPORLASTIC is not aware of any allergic reactions being caused by the material. If you have any problems, please talk to your doctor or the supplier of the aid.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

 Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

 SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

CARE

Before washing, remove the two joint rails from the guide

pockets. We recommend washing GENU-HIT® GS gently in lukewarm water (30°C) with mild detergent and air drying. Do not dry the product in a tumble dryer, on the heating or in the microwave.

Any tension in the knitted fabric that decreases over time will be restored by washing. We therefore recommend washing the brace more frequently.

 **TRANSPORT AND STORAGE CONDITIONS**
Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

MATERIAL COMPOSITION

67% polyamide, 33% spandex

 An online version of the instructions for use can be found on our website at www.sporlastic.de.

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