KASSELER PATELLAR-SEHNENBANDAGE

SPORLASTIC

REF ART.-NR. 07229

KASSELER PATELLAR-SEHNENBANDAGE

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INSTRUCTIONS FOR USE EN

Dear Patient,

We ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

INTENDED USE

KASSELER PATELLAR-SEHNENBANDAGE / KASSELER PATELLAR-SEHNENBANDAGE AKTIV is a proprioception support and exerts pressure on the patella tendon to change the patella movement.

INDICATIONS

- Anterior knee pain
- Patellar tendinitis
- Chondropathia patellae
- Femoropatellare osteoarthritis
- Cruciate ligament insufficiency
- Anterior cruciate ligament

- Post-operative / post-traumatic rehabilitation training: Improved outcome via enhanced and balanced neuromuscular recruitment
- Quadriceps paresis
- Muscular dystrophy

CONTRA-INDICATIONS

- Post-thrombotic syndrome
- In these cases, we recommend wearing the KPSB/-Aktiv only in conjunction with custom-made compression stockings.
- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be supplied.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenic disorders of the sensory system and skin trophism in the body area to be treated (sensory disorders with or without skin damage).

SIDE EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

PUTTING IT ON AND USE

- The KASSELER PATELLAR-SEHNENBANDAGE/KASSELER PATELLAR-SEHNENBANDAGE AKTIV can be worn on the left or right side.

- When correctly fitted, the recesses on the support point upwards.
- The centre of the nubbed silicone pad should rest directly on the patellar ligament below the kneecap.
- Place the support with the pad on the stretched leg and tighten with a slight pull **1**.
- The support should fit without slipping and should feel comfortable to wear.

- If you find the fit too tight after a while, open the support and give 1-2 mm in circumference. Check the fit of the support again for some time and adjust if necessary until the ideal fit is achieved. You will be able to adjust the ideal fit of the support more and more with increasing frequency of use.

- Depending on the physical activity (office, sports, hiking, etc.), the extent of the musculature changes. The support should then be readjusted individually.

- The KASSELER PATELLAR-SEHNENBANDAGE proves its function especially in everyday life and, like the KASSELER PATELLAR-SEHNENBANDAGE AKTIV, can also be used in sports. The neoprene material used in the active version and the fact that it does not have a strap eyelet also allows it to be used in rehabilitation in water or for exercises with ground contact.

⚠ IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.

- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.

- In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sporlastic.de to report the incident.

- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.

- With products involving direct skin contact with an elastomer (e.g. a silicone pad), blistering may develop on the skin if the product is worn too tightly or if it is moved over the skin with high amounts of friction. In this case, we recommend stopping use of the product. Sweat can develop underneath the silicone, which in rare cases can trigger a reaction to your own sweat with corresponding allergic skin changes. SPORLASTIC is not aware of any allergic reactions being caused by the material. If you have any problems, please talk to your doctor or the supplier of the aid.

- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the

doctor in charge of treatment.

- Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

- SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

CARE

We recommend that you wash the KASSELER PATELLAR-SEHNENBANDAGE/ KASSELER PATELLAR-SEHNENBANDAGE AKTIV gently in lukewarm (30°C) water with mild detergent and air dry. Do not dry the product in the tumble dryer, on the heater or in the microwave. Before washing the KASSELER PATELLAR-SEHNENBANDAGE/ KASSELER PATELLAR-SEHNENBANDAGE AKTIV, remove the velcroed silicone pad and close the Velcro fastener of the support. This prevents damage to other items of clothing.

☂☀ TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

- An online version of the instructions for use can be found on our website at www.sporlastic.de.

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