## SPORLASTIC®

REF 07259  
MANU-CAST® ORGANIC D

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## PUTTING IT ON AND USE

- Open the Velcro fasteners of the MANU-CAST® ORGANIC D/MANU-CAST® ORGANIC DP **1** and pull the orthosis over the hand **2**. The anatomically pre-shaped mesh frame lies against the inside of the hand **3**. The movable stell must lie between the thumb and index finger **4**.

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## NOTES FOR THE ORTHOPAEDIC TECHNICIAN

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## INSTRUCTIONS FOR USE EN

Dear patient,  
we ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

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## ACCESORIES/SPARE PARTS:

- Optional available from your specialist dealer: Replacement pads for MANU-CAST® ORGANIC D (REF 17259) and MANU-CAST® ORGANIC DP (REF 17359).
- Removal of the SPORLASTIC 3D SPACE KNIT replacement pad:

- To remove the replacement pad, loosen all reins.
- Detach the replacement pad from the orange clips and remove it.

## INDICATIONS

REF 07259, MANU-CAST® ORGANIC D:

- Conservative/Postoperative
- Damage to the hand in the wrist and metacarpal region
- For immobilisation of the wrist with immobilisation of the fingers and free movement of the thumb

## IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- All indications requiring immobilisation of the wrist with finger fixation, such as:

- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.

- Do not wear the product directly against open wounds.

- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.

- The product can come into contact with splash water.

- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.

- In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sporlastic.de to report the incident.

- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.

- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

- Please return the packaging to the local recycling system. Please observe the local regulations.

- Recycling the plastic frame: Separate the straps/fasteners and the clicked-in upholstery from the frame. Dispose of the frame via your local plastic recycling system.

- Disposal of the textile parts: The Velcro fasteners as well as the padding can be disposed of via your usual household waste.

- SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

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Wszystkie produkty firmy SPORLASTIC GmbH podlegają kontroli w ramach naszego systemu zarządzania jakością.