## MANU-CAST® ORGANIC DP

## SPORLASTIC®

REF 07259

### MANU-CAST® ORGANIC D

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## INSTRUCTIONS FOR USE EN

Dear patient,

we ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

### INTENDED USE

MANU-CAST® ORGANIC D / MANU-CAST® ORGANIC DP is a hand brace for immobilising the wrist, optionally with fixation of the fingers and/or thumb.

### INDICATIONS

REF 07259, MANU-CAST® ORGANIC D:

- Conservative/Postoperative
- Damage to the hand in the wrist and metacarpal region
- For immobilisation of the wrist with immobilisation of the fingers and free movement of the thumb
- All indications requiring immobilisation of the wrist with finger fixation, such as:

- Preoperative
- Post-traumatic
- Carpal tunnel syndrome/CTS
- Inflammation
- Degeneration
- Rheumatic diseases

REF 07359, MANU-CAST® ORGANIC DP:

- Conservative/Postoperative
- Damage to the hand in the wrist and metacarpal region
- For immobilisation of the wrist with immobilisation of the fingers and thumb
- All indications requiring immobilisation of the wrist with finger and thumb fixation, such as:
  - Preoperative
  - Post-traumatic
  - Carpal tunnel syndrome/CTS
- Inflammation
- Degeneration
- Rheumatic diseases
- For paralysis

### CONTRA-INDICATIONS

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be supplied
- Impaired circulation or lymphatic soft tissue swelling
- Neurogenic disorders of the sensory system and skin trophism in the body area to be treated (sensory disorders with or without skin damage)

### SIDE EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

### INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

### PUTTING IT ON AND USE

- Open the Velcro fasteners of the MANU-CAST® ORGANIC D/MANU-CAST® ORGANIC DP **1** and pull the orthosis over the hand **2**. The anatomically pre-shaped mesh frame lies against the inside of the hand **3**. The movable bar must lie between the thumb and index finger **4**.
- Close the Velcro fasteners with measured tension. The wrist strap is thereby wrapped circularly around the wrist and orthosis **5**. The thumb closure of the MANU-CAST® ORGANIC DP can be individually adjusted in position if necessary.
- The MANU-CAST® ORGANIC D/MANU-CAST® ORGANIC DP is correctly tightened when the anatomically pre-shaped mesh frame fits snugly against the palm, wrist and forearm.
- To remove MANU-CAST® ORGANIC D/MANU-CAST® ORGANIC DP, open the circular Velcro straps **6** and pull the orthosis off with the other hand **7**.

### NOTES FOR THE ORTHOPAEDIC TECHNICIAN

- MANU-CAST® ORGANIC D/MANU-CAST® ORGANIC DP can be individually adapted and readjusted as required.
- Immerse the grid frame in hot water (70-80 °C) at the appropriate point for remoulding. The heated area can then be thermoformed.
- Attention: Do not use a hot air blower for postforming, as this leads to uneven heating!
- The buckles of the MANU-CAST® ORGANIC D/MANU-CAST® ORGANIC DP are continuously adjustable and can be adjusted on both sides. The inner belt pads can be positioned freely. The position of the thumb closure can be individually adjusted if necessary.

### SPARE PARTS

Optionally available from your specialist dealer:

- 3D SPACE KNIT frame pad for MANU-CAST® ORGANIC D (REF 17259)
- 3D SPACE KNIT frame pad for MANU-CAST® ORGANIC DP (REF 17359)
- Replacement strap set for MANU-CAST® ORGANIC D and MANU-CAST® ORGANIC DP (REF 37359)

Removing the SPORLASTIC 3DSPACE KNIT replacement pad:

- To remove the replacement pad, loosen all the reins.
- Detach the replacement pad from the orange clips and remove it.

### ACCESSORIES

Optionally available from your specialist dealer:

- MANU-CAST® ORGANIC FX ADD-ON (REF 27459)

### IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- If a significant deterioration in health occurs during the use of the medical device, we ask you to report this serious

incident to your specialist dealer, your doctor or us as the manufacturer, as well as to the responsible authority. You can find our contact information in these instructions for use.

Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.

Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

Please return the packaging to the local recycling system. Please observe the local regulations.

Recycling the plastic frame: Separate the straps/ fasteners and the clicked-in upholstery from the frame. Dispose of the frame via your local plastic recycling system.

Disposal of the textile parts: The Velcro fasteners as well as the padding can be disposed of via your usual household waste.

SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

### CARE

We recommend washing MANU-CAST® ORGANIC D/MANU-CAST® ORGANIC DP gently in lukewarm water (30 °C) with mild detergent and air drying. Do not dry the product in the tumble dryer, on the heating or in the microwave.

### TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

### QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

### MATERIAL COMPOSITION

Pure plastic parts: 100 % polyamide

Textiles: 95 % polyamide, 5 % spandex

An online version of the instructions for use can be found on our website at [www.sporlastic.de](http://www.sporlastic.de)

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