SPORLASTIC®

REF ART.-NR. 07233
MANU-HiT® CARPAL

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The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

PUTTING IT ON AND USE

- The carpal tunnel syndrome brace is available in a left and right version.
- Open all the Velcro fasteners and slip the affected hand into the brace 1.

Move the hand forward in the brace until the recess for the thumb is reached and encloses it at its base 2.

First fix the Velcro tape between the thumb and index finger on the backhand side fleece material 3.

Close the three micro Velcro fasteners and first fix them with a slight pull on the back of the hand 4.

The one-piece CTS-RELIEF-PAD in combination with the volar reinforcement splint lifts the brace sleeve in the area of the carpal tunnel and the median nerve and protects against pressure irritation.

If necessary, correct the fit of the brace and fix the three Velcro straps so that the brace provides stability and a subjectively pleasant wearing comfort.

After checking the optimal fit and position, fasten the straps so tightly that it is comfortable for you.

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Check the functioning of the product with your doctor or specialist retailer.

The product is designed to treat one patient.

Do not wear the product directly against open wounds.

In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.

The product can come into contact with splash water.

Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.

In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sporlastic.de to report the incident.

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With products involving direct skin contact with an elastomer (e.g. a silicone pad), blistering may develop on the skin if the product is worn too tightly or if it is moved over the skin with high amounts of friction. In this case, we recommend stopping use of the product. Sweat can develop underneath the silicone, which in rare cases can trigger a reaction to your own sweat with corresponding allergic skin changes. SPORLASTIC is not aware of any allergic reactions being caused by the material. If you have any problems, please talk to your doctor or the supplier of the aid.

Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

Postoperative immobilisation

Postoperative residual symptoms

Care

We recommend washing MANU-HiT® CARPAL gently in lukewarm water (30°C) with mild detergent and air drying.

• Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be supplied.

• Impaired circulation or lymphatic soft tissue swelling.

• Neurogenic disorders of the sensory system and skin trophism in the body area to be treated (sensory disorders with or without skin damage).

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The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

PUTTING IT ON AND USE

- Used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

An online version of the instructions for use can be found on our website at www.sporlastic.de.

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