
MANU-HiT® POLLEX

SPORLASTIC®

REF ART.-NR. 07645
MANU-HiT® POLLEX

Handgelenkorthese mit Daumeneinschluss
Wrist brace with thumb inclusion
Orthèse de poignet avec inclusion du pouce
Ortesi del polso con inclusione del pollice
Orteza nadgarstka z włączaniem kciuka
Håndledsortose med tommelfingerinddraf-

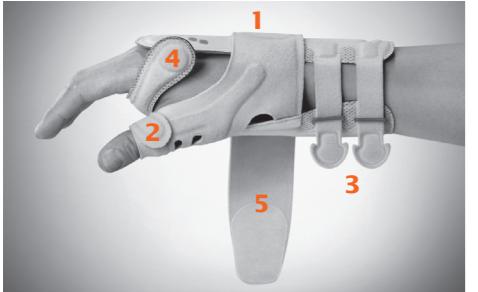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INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

PUTTING IT ON AND USE

- MANU-HiT® POLLEX is available in a left and right version 1.
- Open all Velcro fasteners except for the one on the thumb part and slip into the brace with the affected hand. Guide the thumb into the thumb guide provided on the brace 2.
- Close the Velcro fasteners 3.
- Now insert the Velcro tape between the thumb and index finger and fasten it 4.
- The inelastic wide Velcro strap supplements the stabilisation of the wrist by dosed circular tension 5.
- It may be necessary to readjust the Velcro straps.

NOTES FOR THE ORTHOPAEDIC TECHNICIAN

- The fit of MANU-HiT® POLLEX must be optimally adjusted by pressing the integrated aluminium insert 6 against the thumb, especially at the base joint. By deforming the aluminium insert accordingly, different individual corrective positions of the thumb can be achieved depending on the indication. The deformations can be easily carried out by hand without tools.
- The volar and dorsal reinforcement splint is anatomically pre-shaped for stabilisation of the hand and can be individually adapted if necessary.

IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.

- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.

- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sporlastic.de to report the incident.

- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.

- Product may contain metallic objects. We recommend not putting them in contact with electromagnetic interference.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

- Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

- SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

CARE

We recommend washing MANU-HiT® POLLEX gently in lukewarm water (30°C) with mild detergent and air drying. Do not dry the product in the tumble dryer, on the heating or in the microwave. To avoid damaging other garments during washing, it is best to wash MANU-HiT® POLLEX separately or close the Velcro fasteners in advance and remove the inserted reinforcement rails.

TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

An online version of the instructions for use can be found on our website at www.sporlastic.de.

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