# MANU-HIT® POLLEX CLASSIC

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## INSTRUCTIONS FOR USE EN

Dear Patient,

We ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

### INTENDED USE

MANU-HIT® POLLEX CLASSIC is a wrist brace with thumb fixation for immobilisation.

### INDICATIONS

- Arthrosis of the radial (thumb-side) parts of the carpus
- Arthrosis of the metacarpophalangeal joint of the thumb
- Distortion of the metacarpophalangeal joint of the thumb
- Ulnar collateral ligament lesion of the metacarpophalangeal joint (e.g. ski thumb)
- Painful changes in the scaphoid bone (arthrosis, SNAC-Wrist)
- Radial instability of the wrist, especially the bone bet-

ween the scaphoid and the trapezium/small trapezoid bone

### CONTRA-INDICATIONS

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be supplied.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenic disorders of the sensory system and skin trophism in the body area to be treated (sensory disorders with or without skin damage).

### SIDE EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

### INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

### PUTTING IT ON AND USE

- MANU-HIT® POLLEX CLASSIC is available in a left and right version.
- Open all Velcro fasteners and slip the affected hand into the brace.
- Insert the thumb into the thumb guide provided on the brace until it encloses the metacarpophalangeal joint. Tighten the thumb part. The thumb end joint remains freely movable.
- Thread the two Velcro straps through the eyelets on the forearm and fix the MANU-HIT® POLLEX CLASSIC to the hand, pulling gently at first.
- Then fix the Velcro between your thumb and index finger.
- The non-elastic long Velcro strap is passed in a circular pattern around the wrist and fastened with the desired stabilisation.
- If necessary, adjust the fit of the brace and fix the two Velcro straps on the forearm so that the MANU-HIT® POLLEX CLASSIC provides stability and a subjectively pleasant wearing comfort.

### NOTES FOR THE ORTHOPAEDIC TECHNICIAN

- The dorsal reinforcement splint can be inserted medially or laterally into the guide bar provided, as required.
- This can be removed from the internal insertion openings and individually shaped for an optimal fit on the hand.

### ▲ IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sporlastic.de to report the incident.

- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- Product may contain metallic objects. We recommend not putting them in contact with electromagnetic interference.

- With products involving direct skin contact with an elastomer (e.g. a silicone pad), blistering may develop on the skin if the product is worn too tightly or if it is moved over the skin with high amounts of friction. In this case, we recommend stopping use of the product. Sweat can develop underneath the silicone, which in rare cases can trigger a reaction to your own sweat with corresponding allergic skin changes. SPORLASTIC is not aware of any allergic reactions being caused by the material. If you have any problems, please talk to your doctor or the supplier of the aid.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

- Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

- SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

### CARE

We recommend washing MANU-HIT® POLLEX CLASSIC gently in lukewarm water (30°C) with mild detergent and air drying. Do not dry the product in the tumble dryer, on the heating or in the microwave. To avoid damaging other garments during washing, it is best to wash MANU-HIT® POLLEX CLASSIC separately or close the Velcro fasteners in advance and remove the inserted reinforcement rails.

### ☂️☀️ TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

### QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

- An online version of the instructions for use can be found on our website at [www.sporlastic.de](http://www.sporlastic.de).

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