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## INSTRUCTIONS FOR USE EN

## Dear patient,

We ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

## INTENDED USE

MANU-HIT® / MANU-HIT® KIDS is a brace for immobilising and stabilising the wrist in at least two directions of movement and is used to reduce strain on the carpal bones, joint capsules and ligaments.

## INDICATIONS

- Conservative / postoperative
- Following sprains
- Tendon sheath inflammation
- Arthrosis of the wrist
- Wrist strain symptoms
- Injuries to the carpal bones
- Following removal of plaster casts
- Navicular pseudo-arthritis
- Drop hand
- Lunatomalgia
- Rheumatoid arthritis
- Carpal tunnel syndrome

## CONTRA-INDICATIONS

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be supplied
- Impaired circulation or lymphatic soft tissue swelling
- Neurogenic disorders of the sensory system and skin trophism in the body area to be treated (sensory disorders with or without skin damage)

## SIDE EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

## INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

## PUTTING IT ON AND USE

- Slip your hand into the brace 1.
- Close the Velcro fasteners 2.
- Now insert the Velcro strap 3 between your thumb and index finger and fasten it.
- The inelastic wide Velcro strap 4 completes the stabilisation of the wrist by means of closed circular traction.
- Finally, tighten the two Velcro straps 2 so that MANU-HIT®/MANU-HIT® KIDS sits firmly on your hand without pressing uncomfortably. If necessary, correct Velcro 3 additionally.

## NOTES FOR THE ORTHOPAEDIC TECHNICIAN

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## IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the pre-

sence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.

• Check the functioning of the product with your doctor or specialist retailer.

• The product is designed to treat one patient.

• Do not wear the product directly against open wounds.

• In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.

• The product can come into contact with splash water.

• Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.

• If a significant deterioration in health occurs during the use of the medical device, we ask you to report this serious incident to your specialist dealer, your doctor or us as the manufacturer, as well as to the responsible authority. You can find our contact information in these instructions for use.

• Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.

• Product may contain metallic objects. We recommend not putting them in contact with electromagnetic interference.

• Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

## CARE

We recommend washing MANU-HIT® / MANU-HIT® KIDS gently in lukewarm water (30°C) with mild detergent and air drying. Do not dry the product in the tumble dryer, on the heating or in the microwave. For washing, please remove the two reinforcement rails from the brace. Close the Velcro fasteners before washing. This will keep the Velcro fasteners functional for longer.

## TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

## QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

An online version of the instructions for use can be found on our website at [www.sporlastic.de](http://www.sporlastic.de).

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