MF02810 MOBILAS®

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## en Instructions for use

Dear Patient, This orthopaedic device is classified as a medical device, so we request that you carefully read these Instructions for use. If you have any questions, please contact your attending physician, your nearest specialist dealer or us directly.

#### Intended use

MOBILAS® is a hand positioning system designed to position the hand in a functional position and is used on patients with paralysis that require positioning.

#### Application/indications

- Paralysis requiring positioning, e.g. following a stroke
- Plegia of the hand
- Centrally induced movement restrictions and spasticity in the arm/hand region

#### Contraindications

- Neurogenic sensitivity and trophic disorders of the skin in the affected area

#### Risks/side effects

At present, SPORLASTIC is not associated with any side effects, provided it is used properly and applied correctly. Signs of pressure and damage to blood vessels or nerves may occur nevertheless in connection with fixed devices. Perspiration and skin changes may be observed if there is direct skin contact with an elastomer (e.g. a silicone pad). Please discuss the following with your attending physician before using the support:

- Illnesses in the area of application
- Sensory disturbances, circulatory disorders
- Lymphatic drainage disorders

#### Intended users and target patient group

Patients after assignment and guidance by medical professionals, with due consideration of the indications and contraindications. Healthcare professionals, including carers and relatives who look after the patient and who have received instruction from healthcare professionals.

#### Application and use

- MOBILAS® consists of a wipe-disinfectable base plate (I) with 6 separate press studs and two fixation straps, a ball, a self-adhesive finger rest (II) and a foam part with memory effect (III).
- MOBILAS® comes with an attached pad that can be used to adjust the gliding capability of the base plate on the pad. These components are designed to enable optimised positioning of the patient's hand as required. In addition, the positioning system is built to enable targeted finger and hand exercises.
- Sit at a table or wheelchair table and place MOBILAS® in front of you for positioning and exercise.
- Place the affected hand on the elevation and find a position in which your palm has the greatest possible contact with MOBILAS® ❶
- If your hand wants to move away from the optimum position, use the fixation straps to keep your hand in the desired position ❷
- Attach the straps to the MOBILAS® base plate using the press studs. Do not apply the straps too tightly to avoid constricting the blood circulation.
- The self-adhesive finger rest made of skin-friendly silicone can be positioned on the plastic of the support disc as often as required and also removed again. Clean with soap and water to restore the adhesive strength if necessary. The silicone finger rest enables positioning and stabilisation of the fingers on the support disc and is used to hold the DIP joints if necessary ❸
- Use the foam parts with memory effect to adjust the hand positioning to suit your needs. Fingers with mild to moderate spasticity are supported dynamically. The foam yields slightly when the tone increases and returns to its original position when the tone decreases ❹
- The gliding base also turns MOBILAS® into a gliding mobilisation system. To use in this way, unclip the cover on the back of MOBILAS® and turn it to the side without adhesive coating ❺
- The ball turns MOBILAS® into a mobilisation gyroscope.

Scan the QR code on the packaging to access the digital exercise manual.

#### Reuse <sup>(1)</sup>

MOBILAS® is intended exclusively for the individual treatment of a single patient.

#### Important notes <sup>(1)</sup>

- As a rule, the medical use and function of an orthopaedic device, as well as how long it is worn, should always be discussed with the attending physician or specialist retailer.
- Kindly refer also to the symbols on the packaging and these Instructions for use.
- In order to ensure a long service life, the product must not be used in conjunction with fatty and acidic products, ointments or lotions.
- The MOBILAS® base plate is wipe-disinfectable.
- Do not wear the product on open wounds.
- If included, always ensure that all Velcro fasteners are properly closed. Damage caused by improper closing of the Velcro fasteners shall not constitute grounds for complaint.
- Consult with your attending physician or specialist dealer if you notice a deterioration in your state of health while using this device.
- The device may contain metallic objects. Patients must therefore take off the device prior to an MRI scan.

#### Disposal <sup>(1)</sup>

Please comply with local regulations when disposing of the packaging and product after use.

#### Care and maintenance

We recommend that you gently wash the textile components of the Mobilas® in lukewarm water (max. 40 °C) with mild detergent and then leave to dry without direct exposure to sunlight. Do not dry the product in a tumble dryer, on the radiator or in the microwave. The plastic parts can be cleaned with warm water and soap and are wipe-disinfectable.

<sup>(1)</sup> The product is essentially maintenance-free with the correct handling and care.

#### Transport and storage conditions <sup>(1)</sup>

Store the product at room temperature and away from moisture and direct sunlight.

#### Complaints

Kindly contact your specialist dealer if you have any complaints about our product. If you return the product to us, please clean it beforehand.

#### Notification obligations

Please report any physical damage due to a product defect to SPORLASTIC and your national authority for the supervision of medical devices. You will find our SPORLASTIC contact details at the end of these instructions for use.

#### Disclaimer

Please use the medical device only as discussed with your attending physician or specialist dealer. No liability is accepted for self-medication. Unprofessional modifications to the product and/or improper use of the product shall void any manufacturer's product liability.

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<sup>(1)</sup> An online version of these Instructions for use is available on our website at [www.sporlastic.de](http://www.sporlastic.de).

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