# OLECRANON-BANDAGE

**SPORLASTIC®**

REF ART.-NR. 07235  
OLECRANON-BANDAGE

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## INSTRUCTIONS FOR USE EN

### Dear Patient,

We ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

### INTENDED USE

The OLECRANON-BANDAGE is an elbow compression support with pad for compression and relief of the elbow joint.

### INDICATIONS

- Chronic olecranon bursitis
- After contusion with persistent irritation in the region of the olecranon
- Insertion tendopathy of the triceps tendon
- Condition after surgery (bursectomy)
- Prophylaxis of olecranon bursitis

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### INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

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### IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should

be discussed with the doctor who is in charge of your treatment.

- Check the functioning of the product with your doctor or specialist retailer.

The product is designed to treat one patient.

Do not wear the product directly against open wounds.

In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.

The product can come into contact with splash water.

Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.

In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sporlastic.de to report the incident.

Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.

With products involving direct skin contact with an elastomer (e.g. a silicone pad), blistering may develop on the skin if the product is worn too tightly or if it is moved over the skin with high amounts of friction. In this case, we recommend stopping use of the product. Sweat can develop underneath the silicone, which in rare cases can trigger a reaction to your own sweat with corresponding allergic skin changes. SPORLASTIC is not aware of any allergic reactions being caused by the material. If you have any problems, please talk to your doctor or the supplier of the aid.

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