SPORLASTIC®

REF ART.-NR. 07605
RHIZO-HiT® CLASSIC

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INSTRUCTIONS FOR USE EN

Dear Patient,

We ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

INTENDED USE

RHIZO-HiT® CLASSIC is a thumb brace for immobilisation of the thumb metacarpophalangeal joint and thumb saddle joint with free mobility of the thumb end joint.

INDICATIONS

- Injuries, ligament instabilities in the area of the thumb metacarpophalangeal and thumb saddle joint (e.g. ski thumb)
- Arthritis of the radial carpus of the MCP 1
- Rhizarthrosis

CONTRA-INDICATIONS

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be supplied.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenic disorders of the sensory system and skin trophism in the body area to be treated (sensory disorders with or without skin damage).

SIDE EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

It is possible to shorten the thumb closure strap if it is too long.

It is possible to adjust the tension of the strap in such a way that you continue to feel comfortable while stabilising the thumb and wrist.

For easy entry into RHIZO-HiT® CLASSIC, leave the Velcro straps in their position on the thumb end tube or in the deflection eyelet when putting it on repeatedly.

Fig. 3 shows the correct fit of RHIZO-HiT® CLASSIC.

PUTTING IT ON AND USE

- RHIZO-HiT® CLASSIC can be worn on the left or right side.
- Velcro the end of the thumb tube so that the opening for your thumb can be grasped as tightly as possible but is still comfortable to wear.
- To achieve an optimal fit on the thumb, the hook strap should be looped as precisely as possible over the fleece.
- Slip the affected thumb into the brace 1.

- Grasp the non-elastic Velcro strap for fixing RHIZO-HiT® CLASSIC to the wrist with your thumb and index finger and thread it into the deflection eyelet 2.

- The loop allows a controlled pull of the Velcro strap with little effort to stabilise the wrist.

- Adjust the tension of the strap in such a way that you continue to feel comfortable while stabilising the thumb and wrist.

- For easy entry into RHIZO-HiT® CLASSIC, leave the Velcro straps in their position on the thumb end tube or in the deflection eyelet when putting it on repeatedly.

- Fig. 3 shows the correct fit of RHIZO-HiT® CLASSIC.

NOTES FOR THE ORTHOPAEDIC TECHNICIAN

- It is possible to shorten the thumb closure strap if it is too long.

IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.

- Check the functioning of the product with your doctor or specialist retailer.

- The product is designed to treat one patient.

- Do not wear the product directly against open wounds.

- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.

- The product can come into contact with splash water.

- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.

- In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sporlastic.de to report the incident.

- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.

- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

- Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

- SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

CARE

We recommend washing RHIZO-HiT® CLASSIC gently in lukewarm water (30°C) with mild detergent and air drying. Do not dry the product in the tumble dryer, on the heating or in the microwave.

TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

An online version of the instructions for use can be found on our website at www.sporlastic.de.

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