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INSTRUCTIONS FOR USE EN

Dear patient,

We ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

INTENDED USE

SWEDE-O-UNIVERSAL is an ankle brace for stabilising the upper and lower ankle joint.

INDICATIONS

- Conservative treatment of ankle sprains of all degrees of severity
- Postoperative treatment
- Prophylaxis of distortions

CONTRA-INDICATIONS

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be supplied.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenic disorders of the sensory system and skin tropism in the body area to be treated (sensory disorders with or without skin damage).

SIDE EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

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side of the ankle, in the pockets provided **1**.

- Start tying the two lowest eyelets comfortably tight.

Important for ankle stability:

Lace the next three eyelets, which are close together, as tightly as possible to achieve the best possible stability effect.

- The remaining eyelets are laced with a pleasantly firm pull of the laces.

When the SWEDE-O-UNIVERSAL ankle brace is completely put on, check the distance between the inner lacing strips. If the gap is larger than 2 cm, the brace is too small. If the space is less than 0.5 cm, the brace is too large. Please select the appropriate size.

- In case the laces become loose during (sports) activities, tie them tightly again, especially at the three eyelets that are close together.

NOTES FOR THE ORTHOPAEDIC TECHNICIAN

- In order to achieve an optimal fit as well as optimal stability for the ankle joint, the reinforcement inserts used for the malleoli should be thermoplastically adapted to the individual anatomy.

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IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.

- Check the functioning of the product with your doctor or specialist retailer.

The product is designed to treat one patient.

- Do not wear the product directly against open wounds.

In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.

The product can come into contact with splash water.

Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.

In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sportastic.de to report the incident.

Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.

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Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

CARE

Remove the reinforcement inserts from the side pockets before washing.

We recommend washing SWEDE-O-UNIVERSAL gently in lukewarm water (30°C) with mild detergent and air drying. Do not dry the product in the tumble dryer, on the heating or in the microwave.

TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

An online version of the instructions for use can be found on our website at www.sporlastic.de.

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