<b>LOT</b>	DE	Fertigungslosnummer, Charge
	EN	Production batch number, batch
	FR	Numéro de lot de fabrication
	IT	Numero di lotto di produzione, partita
	PL	Numer partii produkcyjnej, szarża
	DA	Batchnummer, batch
<b>PZN</b>	DE	Pharmazentralnummer
	EN	Pharmaceutical registration number
	FR	Numéro central de la pharmacie
	IT	PZN (Germania)
	PL	Numer centralny farmacji
	DA	Farmacentralnummer
LV	"Pharma-Zentral" numurs	
EL	Φαρμακευτικός κεντρικός αριθμός	

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# INSTRUCTIONS FOR USE EN

## INTENDED USE

VERTEBRADYN® FORCE is a bridging brace for relief and correction of the lumbar spine through de-lordosis.

## INDICATIONS

- Conservative/post-operative

All indications requiring relief of the lumbar spine through reduction of the degree of lordosis, such as:

- Severe lumbosciatica with muscular deficits on a background of significant disc protrusions/prolapse without absolute indication for surgery
- Severe radicular, pseudoradicular lumbar syndrome
- Spondylolisthesis with recurrent lumbosciatica
- Severe lumbar deformity with facet joint syndrome/arthritis
- Spinal canal stenosis with short walking distance without significant pareses
- Inflammation, degeneration

All indications requiring relief of the posterior edge of the vertebra, such as:

- Post-traumatic small vertebral fractures
- Pathological fractures

## CONTRA-INDICATIONS

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be treated.
- Impairment of circulation or lymphatic soft tissue swelling.
- Neurogenic disorders of the sensory system or skin trophism in the part of the body being treated (sensory disturbances with and without skin damage).

## SIDE-EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

## INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

## PUTTING IT ON AND USE

- We recommend wearing a thin, tight-fitting T-shirt under the product.
- Loosen the straps and guide your hands into the hand slings at the front.
- Place the brace around your body and position the back section centrally over the spine. The orange-coloured lower edge of the brace ends above the sacrum. The frame extends from the lower ribs to the waist.
- Close the front fastening so that the brace builds up comfortable pressure. Optionally, you can use the abdominal late by affixing it between the fastenings.
- Tighten the two lower tension elements evenly on the abdominal fastening. Then tighten the upper tension elements.
- The lower fastener on the front should run flush and parallel with the thigh when sitting and not exert any pressure on the crease of the groin.

⚠ VERTEBRADYN® FORCE requires customised and skilled adaptation to the patient by trained personnel. This is the only way to ensure its full performance and maximum wearing comfort.

## INFORMATION FOR ORTHOPAEDIC TECHNICIANS

- The height of the brace frame can be adjusted.
- To do this, use a 2.5cm Allen key, open the corresponding screws **1** and adjust the height **2**.
- The wings of the lower part of the frame should lie between the iliac crest and the greater trochanter **3**. The wings of the upper part of the frame should enclose the lower rib **4**.
- Once the height is correctly adjusted, fasten the screws on the orthotic frame **5** at the appropriate points.
- The back frame is made of hard aluminium and can be shaped by hand or with a bending iron **6**.
- The spring steel bars can be individually shaped.
- VERTEBRADYN® FORCE extends from the anterior costal arch to the symphysis.
- For increasing mobilisation, the brace can be gradually removed in consultation with the treating physician. To do this, the frame or abdominal plate can be removed from the brace.
- The brace assumes the function of a bridging brace.
- By removing the abdominal plate and frame, VERTEBRADYN® FORCE becomes a lumbar support brace.

## ⚠ IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.

- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at [info@sporlastic.de](mailto:info@sporlastic.de) to report the incident.
- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- Product may contain metallic objects. We recommend not putting them in contact with electromagnetic interference.
- With products involving direct skin contact with an elastomer (e.g. a silicone pad), blistering may develop on the skin if the product is worn too tightly or if it is moved over the skin with high amounts of friction. In this case, we recommend stopping use of the product. Sweat can develop underneath the silicone, which in rare cases can trigger a reaction to your own sweat with corresponding allergic skin changes. SPORLASTIC is not aware of any allergic reactions being caused by the material. If you have any problems, please talk to your doctor or the supplier of the aid.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

## CARE

We recommend washing VERTEBRADYN® FORCE gently in hand-hot water (30°C) with a fine detergent and leaving it to dry naturally. Do not dry VERTEBRADYN® FORCE in a tumble drier, on a radiator or in the microwave. Remove the frame before washing. The cover can be washed at the same time. Close the Velcro fasteners when washing so that they remain functional for longer and prevent damage to other items of clothing.

Tension, which declines over the period of wear, is restored to the shaped knitted fabric by washing it. We therefore recommend washing the brace frequently.

## TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

## QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

An online version of the instructions for use can be found on our website at [www.sporlastic.de](http://www.sporlastic.de).

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