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INSTRUCTIONS FOR USE EN

Dear Customer,

We ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

INTENDED USE

VERTEBRADYN® SUPREME SENSO is a brace for stabilising the lumbar spine with a nubbed SPORLASTIC AIR MATRIX silicone friction pad.

INDICATIONS

- Conservative
- All indications where stabilisation of the lumbar spine by pressure distribution with a back pad is necessary, such as:
 - Lumbaralgia

- Muscular imbalances
- Radicular irritation syndromes
- Spinal canal stenosis
- Preventive stabilisation

CONTRA-INDICATIONS

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be supplied.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenic disorders of the sensory system and skin trophism in the body area to be treated (sensory disorders with or without skin damage).

SIDE EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

PUTTING IT ON

- We recommend wearing a thin, tight-fitting T-shirt under the product.
- Attach the heart-shaped SPORLASTIC AIR-MATRIX silicone friction pad with the mesh cover to the Velcro velour on the inside of VERTEBRADYN® SUPREME SENSO. The narrow side of the pad faces the lower edge of the brace and the nap faces the body. The correct positioning of the brace with the pad attached is achieved when you hold the two ends of the brace apart behind your back in such a way that the back pad comes to rest centrally over the lumbar spine **1**.
- Pass your fingers through the loops attached to the two front ends of the brace **2**.
- When closing, make sure that both ends of the brace are taped parallel to each other. Tighten the two ends of the brace slightly and place them, right over left, on top of each other and Velcro them **3**.
- Pull the fingers out of the loops and fasten the clasp completely **4**.
- VERTEBRADYN® SUPREME SENSO should fit snugly with high compression so that the combination of the pad in the lumbar region and the abdominal support can provide significant relief for the spine. When fitting with high compression, make sure that it is comfortable to wear.
- The front lower end of the VERTEBRADYN® SUPREME SENSO should be flush and parallel to the thigh when seated and should not cause pressure in the groin **5**.
- To take off the brace, open the Velcro fastener **6**.

⚠ IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.

- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sporlastic.de to report the incident.
- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- With products involving direct skin contact with an elastomer (e.g. a silicone pad), blistering may develop on the skin if the product is worn too tightly or if it is moved over the skin with high amounts of friction. In this case, we recommend stopping use of the product. Sweat can develop underneath the silicone, which in rare cases can trigger a reaction to your own sweat with corresponding allergic skin changes. SPORLASTIC is not aware of any allergic reactions being caused by the material. If you have any problems, please talk to your doctor or the supplier of the aid.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

CARE

We recommend washing VERTEBRADYN® SUPREME SENSO gently in lukewarm water (30°C) with mild detergent and air drying. Do not dry the product in the tumble dryer, on the heating or in the microwave.

To avoid damaging other items of clothing during washing, it is best to wash the brace separately or close the Velcro fasteners beforehand.

Any tension in the knitted fabric that decreases over time is restored by washing. We therefore recommend washing the brace more frequently.

TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

MATERIAL COMPOSITION

12 % elastane, 88 % polyamide

An online version of the instructions for use can be found on our website at www.sporlastic.de.

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